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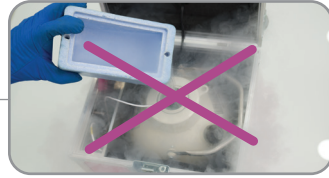
Do not refill the tank with liquid nitrogen.

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Do not handle the temperature sensor or the gauges or switch off the box electronics.

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Do not open or remove the enclosed documentation on the outside of the transport boxes.

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INSTRUCTIONS AND PROTOCOL FOR TANK RECEPTION

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Make sure the box is addressed to your Clinic checking the label with the address. If so, cut the external cable tie and open the box.

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Open the envelope that is inside the box and make sure that the report corresponds with the order made. If that is the case, cut the container cable tie.

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Open the container, fill in the goblets without removing them from the container canister and check that its content matches the delivery report.

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If any of the prior steps are not correct, please notify us before devitrifying/thawing the biological material.

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Pass the rod and goblet containing cryotops to your storage tank and remove the rest of liquid nitrogen that may be left inside the container.

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In the documents attached to the delivery you will find two reports, fill one out and sign it, sending it back in the box or once scanned, by e-mail.

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When returning the container, lock the metallic box using the white cable tie that you will find inside.

4

↓

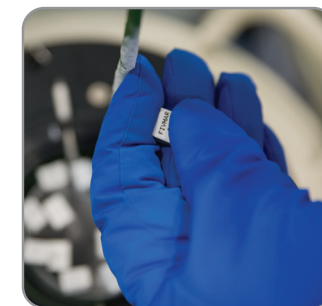
1

↓

2

↓

2

+34 662 154 575